

## General Information

The Days Healthcare No. 203A 'Atlas' Walking Frame is adjustable in height and fitted with arthritic attachments. It is designed to give the user maximum support. Fitted with four castors (two brake castors at the rear) which enables it to travel in any direction.

## Adjustments

To adjust the overall height of the frame, loosen the hand knobs, adjust the height of the armrest making sure both sides are at the same height then tighten the hand knobs.

To adjust the axilla pads loosen the hand knobs and pull up to a comfortable height for the user then re-tighten the hand knobs.

The handles can be adjusted to suit the individual user by loosening the hand knob underneath and moving the handle to a comfortable position. After adjustment ensure the hand knob is securely re-tightened.



203A Atlas Walking Frame  
with Arthritic Attachments

## Maintenance

The Days Healthcare 'Atlas' Walking Frames are designed for minimum maintenance. They require the occasional wipe-over with a soft cloth. Use general purpose neutral detergent and warm water. Do not use harsh abrasive materials!

To disinfect clean as above and wipe over with a solution of 1,000 parts per million available chlorine disinfectant (haz tabs/precept), rinse and dry. We do not recommend any decontamination process that involves immersion.

Daily check that the castors turn freely and there is no debris building around the axle.

Check that the front hand knobs are tight and holding the frame rigid.

Check that the axilla pads rotate freely but do not pull off. To adjust, use the hand wheels underneath the pads.

Check that the brake levers stop the rear castors from swivelling and the wheels from turning.

Periodically check that all screws, nuts and bolts are securely fastened. Check moving parts for wear.

## Specifications

Height Adjustment: 94 - 142 cm (37" - 56")

Overall Width/Depth: 59/76 cm (23¼"/ 30")

Max. user weight: 160 kg (25 stone)

## Warranty Information

All Days Healthcare walkers are warranted for one year from the date of purchase against faulty workmanship or materials. Please contact your supplier/dealer should a fault occur.

The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses, etc.

The manufacturer will not accept responsibility for any damage or injury caused by misuse or non-observance of the instructions set out above.

*Days Healthcare Ltd. is not responsible for any damage or injury due to improper or unsafe use of their Walking Aids.*

