

# 539L LIGHTWEIGHT SHOWER STOOL

## Introduction

The Days Healthcare 539L Shower Stool is a lightweight, height adjustable shower stool manufactured from aluminium tubing and fitted with a plastic seat. It has built-in side handles which offer stability and additional support to the user.



## Height Adjustment

To adjust the height grasp the 'snap lock' (see fig 1) between the thumb and forefinger and pull it out. Slide the leg in or out to the required position. Push the clip back firmly into the hole. Repeat the same procedure for the other legs making sure all four legs are adjusted to the same height.

## Seat Adjustment

The plastic seat is a clip on design and is secured in position to the frame cross tubes.



Snap Lock

## Cleaning

The Days Healthcare 539L Shower Stool is designed for minimum maintenance .

To clean, wipe over with a wet soft cloth. Use general purpose neutral detergent and warm water. The shower chair may be placed under running water at low pressure or showered down to assist cleaning. The plastic seat can be unclipped to make cleaning easier.

Do not use harsh abrasives or bleach based cleaners to avoid discolouration.

To disinfect clean as above and wipe over with solution of one thousand parts per million available chlorine disinfectant (haztabs/precept), rinse and dry.

## Maintenance

Do not store in sub-zero temperature. Periodically check for elongated or stressed adjustment holes. Check for split, worn or loose rubber ferrules and Snap Locks.

Note - Do not use with a damaged or split seat.

## Specification

Model	Overall Width	Overall Depth	Seat Width	Seat Depth	Seat Height	Max. User Weight
539L	550 mm	380 mm	370 mm	280 mm	460 - 590 mm	160 Kg

## Spare Parts

Spare Parts No.	Description
539LPL001	Clip on Plastic Seat

## Warranty Information

The Days Healthcare model no. 539L is warranted for one year from date of purchase due to faulty workmanship and materials. Please contact your supplier/dealer should a fault occur. The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses etc. The manufacturer will not accept responsibility for any damage or injury caused by misuse or non observance of the instructions set out above.

*Days Healthcare Ltd. is not responsible for any damage or injury due to improper or unsafe use of their Shower Stools.*

